


# DRSHD™ 1080p


High Definition Image Management Solution

## Photos and Videos

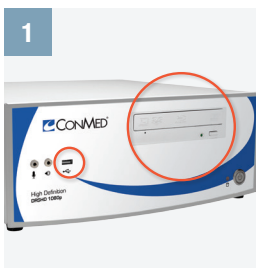


### BEFORE THE CASE BEGINS

Insert **USB memory card** into card slot or a Blu-Ray® disk into the tray.

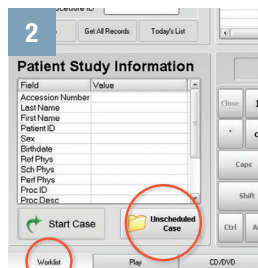


### STARTING

Press the **Worklist** tab.  
Press **Unscheduled Case**.

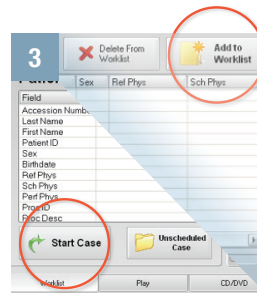


### ADMIT PATIENT

Enter patient information.  
Press **Start Case**.  
Press **Add to Worklist** to add the patient to the local worklist.

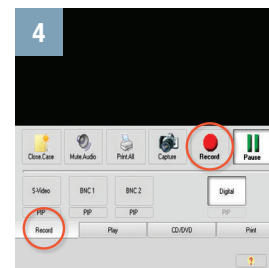


### RECORDING

In the **Record** tab.  
Tap **Capture** for photos.  
Tap **Record** to begin and **Pause** to end videos.

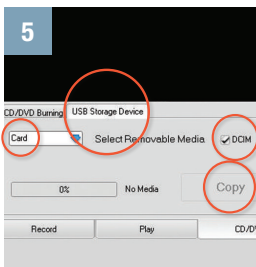


### SAVE TO CARD

In the **CD/DVD** then **USB Storage Device** tabs.  
Dropdown and select **Card**  
If going to iPad®, check **DCIM**.  
Select desired images and tap **Copy**.

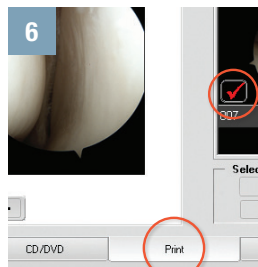

### PRINTING

In the **Print** tab **Select images**, or select **All**.  
Tap **Print**.  
Auto Printing options in User Guide.

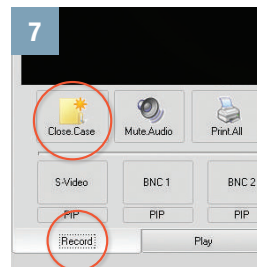

### CLOSE CASE

Closing a case automatically prints all remaining photos.  
In the **Record** tab.  
Tap the **Close Case**.

For saving images and videos to iPad®, see the back of this page.

Connect with **Product Support** for the most efficient and convenient answer to your ConMed product question.

800-237-0169, option 3  
prodspecialist@linvatec.com

## DRSHD™ 1080p

High Definition Image Management Solution

### Saving to iPad®

To begin a case, see the Quick Start Guide for photos and videos on the back of this page.

It is important that when the option for **SAVE TO CARD** is chosen that **DCIM** is checked when saving for use on an iPad® (see quick start guide).

#### AFTER THE CASE IS CLOSED

Remove the SD memory card from the DRSHD 1080p.

Insert the card into the Apple Lightning™ to SD Card Camera Reader.



#### IMPORT TO iPad®

The iPad® Photos app will automatically open and you will have the option to select or import all images to the iPad®.

After you import, **select delete** the images from the SD card.



ios6 and below



ios7

#### ACCESSORIES REQUIRED

##### Apple iPad®


Other tablets are compatible but may require different accessories.

##### Apple Lightning™ to SD Card Camera Reader

##### SD Memory Card

There are many options. Class 10 Memory Card 45MB/s or faster recommended.

*(32GB will hold approximately 30 minutes of High Quality HD 1080p video.)*

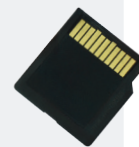

##### USB to SD Card Reader

There are many options.



#### ORGANIZE

Transfer images to the iPad® after each surgery (each “event”).

In the iPad® Photos app you have the option to view by Events.

This is the easiest option for viewing individual patients.



Connect with **Product Support** for the most efficient and convenient answer to your ConMed product question.

**800-237-0169, option 3**  
[prodspecialist@linvatec.com](mailto:prodspecialist@linvatec.com)