





## Eliminator® PET Balloon Dilators

ConMed's Eliminator PET® Balloon Dilators offer a simple solution to dilation even in the tightest of strictures. Constructed of patented PET material, these dilators deploy a multi-stage inflation technique eliminating the need for more than one balloon.

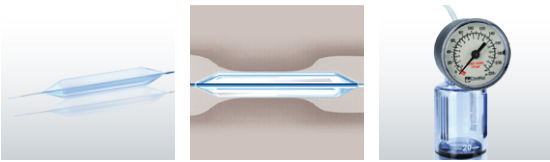

**NAKAO SPIDER-NET™**  
RETRIEVAL DEVICE

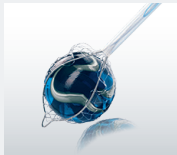


**AMERICAN DILATION**  
SYSTEM



**AMERICAN DILATION**  
MARKED SPRING TIP GUIDEWIRE




To learn more about these and other innovative products, call **800-448-6506** or visit **ConMed.com**.

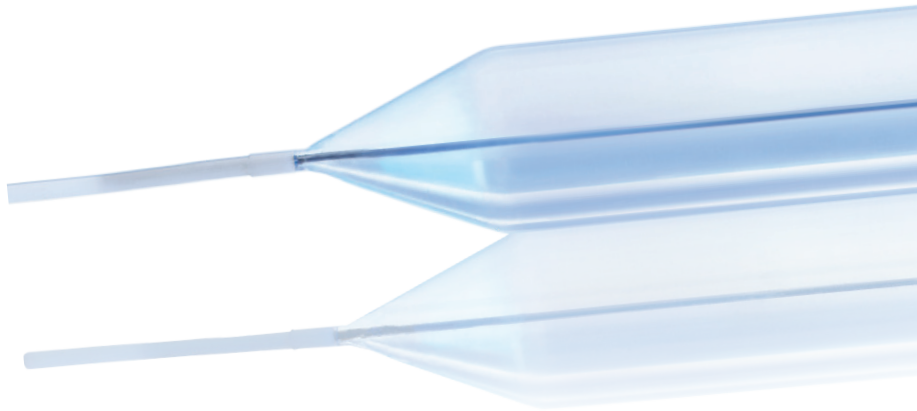


## Pressure Dilates Strictures, Not Size

Balloons dilate strictures with the radial force generated by inflating them to predetermined pressures. The Eliminator® balloons patented PET material disperses pressure along the full length of the balloon ensuring repeatable, reliable balloon sizing. When resistance is met, with non PET balloons, the balloon may expand into the area surrounding the stricture. When this occurs, the expected balloon diameter may not be achieved when inflated to recommended levels.

**In contrast, Eliminator PET Balloon Dilators are true to size, every time**

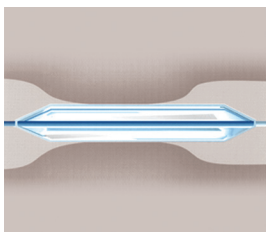

ELIMINATOR PET  
BALLOON DILATOR

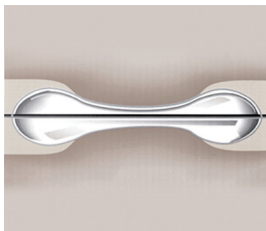


COMPETITOR NON PET  
BALLOON DILATOR

Balloon Size	Stage One	Stage Two	Stage Three*
6mm /18F	60psi/4.1atm	90psi/6.1atm	120psi/8.2atm
8mm /24F	60psi/4.1atm	90psi/6.1atm	120psi/8.2atm
10mm /30F	45psi/3.1atm	68psi/4.6atm	90psi/6.1atm
12mm /36F	45psi/3.1atm	68psi/4.6atm	90psi/6.1atm
15mm /45F	30psi/2.0atm	45psi/3.1atm	60psi/4.1atm
18mm /54F	25psi/1.7atm	38psi/2.6atm	50psi/3.4atm
20mm /60F	25psi/1.7atm	38psi/2.6atm	50psi/3.4atm

## The Eliminator® Balloon Advantage

- Accurate Sizing at Maximum Pressure
- No Waisting
- Valued Performance

\*Maximum recommended inflation pressure

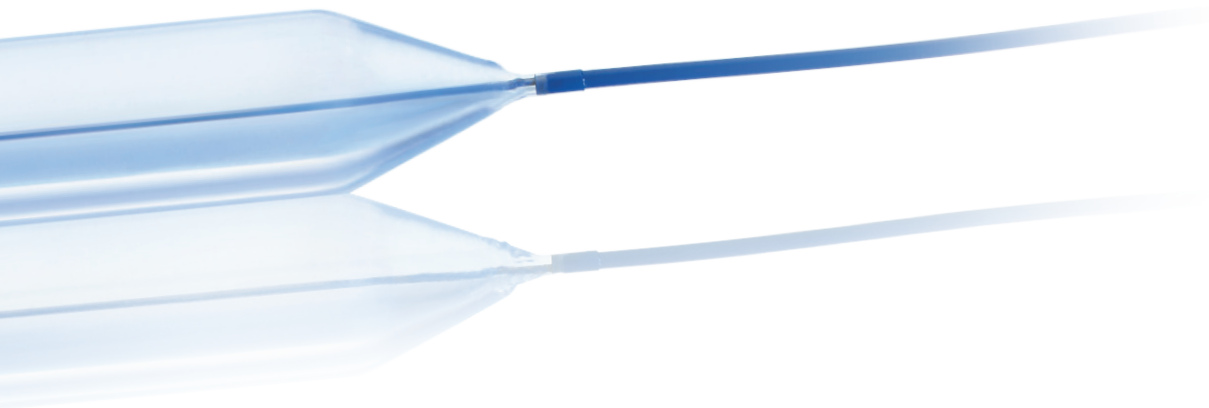


## ConMed's Rule of One

The ConMed Eliminator® PET Balloon Dilators combine the radial strength you need to dilate even the tightest strictures with an innovative and cost-effective alternative. The Multi-Stage Inflation Technique is a simple method for obtaining a complete and reliable dilation with a single Eliminator PET Balloon Dilator.

## A Cost-Effective Approach

The Multi-Stage Inflation Technique helps you control the cost of balloon dilation. Procedure product cost may be reduced compared to the traditional three-stage dilation procedure. Eliminator PET Balloon Dilators are more cost-effective than the competitive serially expanding balloon dilator. When the premium price is factored in, the per procedure product cost is approximately 46% lower when utilizing the Eliminator balloon.

### Approximate Cost Per Procedure

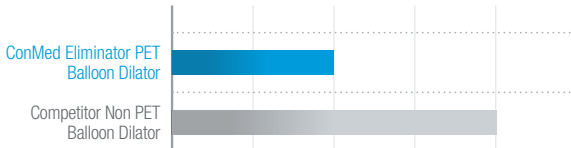

ConMed reaffirms its commitment to providing simple, cost-effective solutions with a Multi-Stage Inflation Technique. ConMed's high-quality, high-value products help our customers control costs and increase product efficiencies. Known for commitment to customer service and support, ConMed is the only company in the industry that offers written product performance guarantees.



BALLOON INFLATION DEVICE



525 French Road, Utica, NY 13502

ELIMINATOR®  
PET BALLOON  
DILATORS

## Ordering Information

Description	Balloon Diameter	Balloon Length	Catheter Length	Unit	Catalog Number
<b>Eliminator® PET Biliary Balloon Dilators</b>					
Esophageal Balloon Dilator	6mm	8cm	180cm	1/box	000340
Esophageal Balloon Dilator	8mm	8cm	180cm	1/box	000341
Esophageal Balloon Dilator	10mm	8cm	180cm	1/box	000342
Esophageal Balloon Dilator	12mm	8cm	180cm	1/box	000343
Esophageal Balloon Dilator	15mm	8cm	180cm	1/box	000344
Esophageal Balloon Dilator	18mm	8cm	180cm	1/box	000345
Esophageal Balloon Dilator	20mm	8cm	180cm	1/box	000380
Esophageal Advantage Pack (5ea. of No.000343)	12mm	8cm	180cm	5/box	000443
Esophageal Advantage Pack (5ea. of No.000344)	15mm	8cm	180cm	5/box	000735
Esophageal Advantage Pack (5ea. of No.000345)	18mm	8cm	180cm	5/box	000736
Esophageal Advantage Pack (5ea. of No.000380)	20mm	8cm	180cm	5/box	000379
Pyloric/Colonic Balloon Dilator	6mm	4cm	230cm	1/box	000844
Pyloric/Colonic Balloon Dilator	8mm	4cm	230cm	1/box	000845
Pyloric/Colonic Balloon Dilator	10mm	4cm	230cm	1/box	000846
Pyloric/Colonic Balloon Dilator	12mm	4cm	230cm	1/box	000847
Pyloric/Colonic Balloon Dilator	15mm	4cm	230cm	1/box	000848
Pyloric/Colonic Balloon Dilator	18mm	4cm	230cm	1/box	000849
Pyloric/Colonic Balloon Dilator	20mm	4cm	230cm	1/box	000861
Pyloric/Colonic Advantage Pack (5ea. of No.000847)	12mm	4cm	230cm	5/box	000857
Pyloric/Colonic Advantage Pack (5ea. of No.000848)	15mm	4cm	230cm	5/box	000858
Pyloric/Colonic Advantage Pack (5ea. of No.000849)	18mm	4cm	230cm	5/box	000859
Pyloric/Colonic Advantage Pack (5ea. of No.000861)	20mm	4cm	230cm	5/box	000867
<b>Inflation Device</b>					
ConMed Balloon Inflation Device				1/box	100851

Customer Service: 1-800-448-6506  
International Sales: 1-315-797-8375  
Fax: 1-800-438-3051

ConMed.com  
info@mail.conmed.com