



CALCANEAL LENGTHENING  
& OSTEOTOMY WEDGES

*Pre-Shaped Allograft Wedge Solutions  
for Deformity Correction*

**ENHANCE™**  
*Biologic Solutions for Extremities*


**CONMED™**  
L I N V A T E C

## ALLOGRAFT WEDGES

Pre-Shaped Wedges from ConMed Linvatec provide surgeons with convenient and reliable allografts for flatfoot deformity correction.

- Dense cancellous bone provides an excellent reincorporation environment
- Two-side cortical rim provides structural strength
- Pre-sized design eliminates the need for intraoperative shaping
- Cotton and Evans technique specific shapes

### EVANS WEDGE

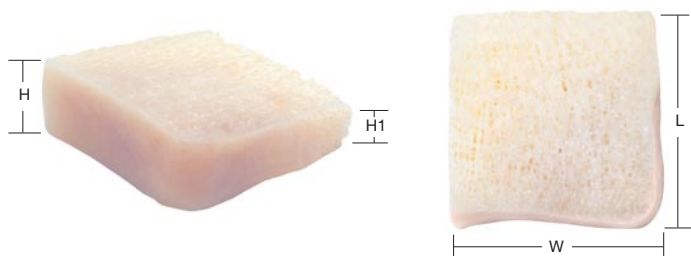

H	H1	L	W
6mm	4mm	22mm	22mm
8mm	5mm	22mm	22mm
10mm	5mm	22mm	22mm
12mm	5mm	22mm	22mm
14mm	6mm	22mm	22mm

### COTTON WEDGE



H	L	W
5mm	22mm	16mm
6mm	22mm	16mm
7mm	22mm	16mm
8mm	22mm	16mm


### ARCH WEDGE




H	L	W
5mm	22mm	16mm
6mm	22mm	16mm
7mm	22mm	16mm
8mm	22mm	16mm

Note: Allograft wedges may be trimmed from the cancellous sides to customize length and width.

## INSTRUMENTATION

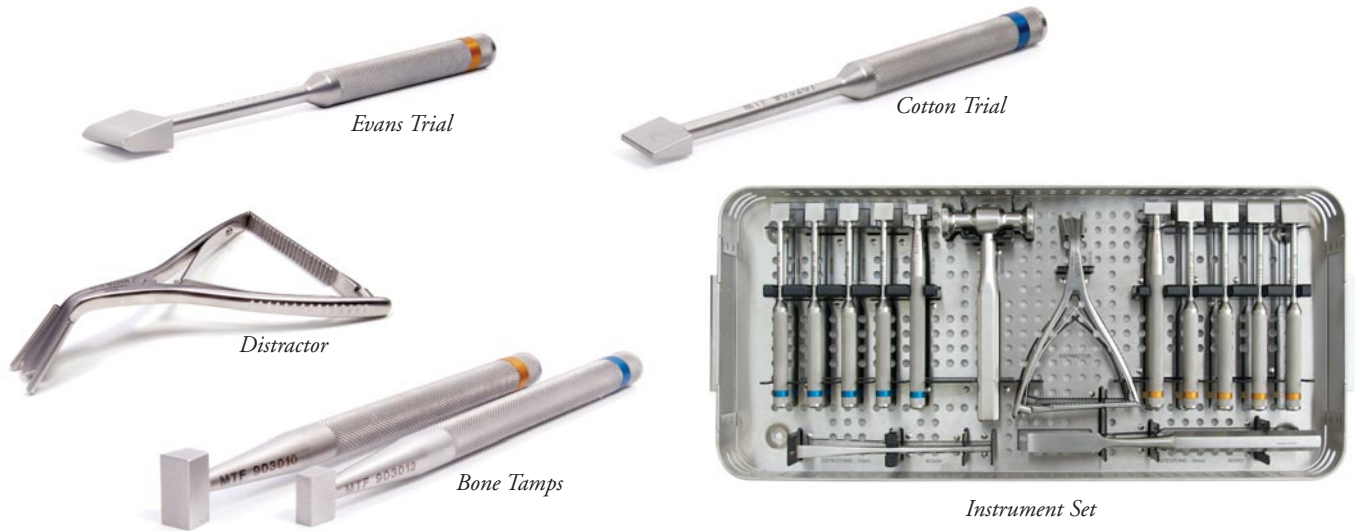

## CALCANEAL LENGTHENING PROCEDURE

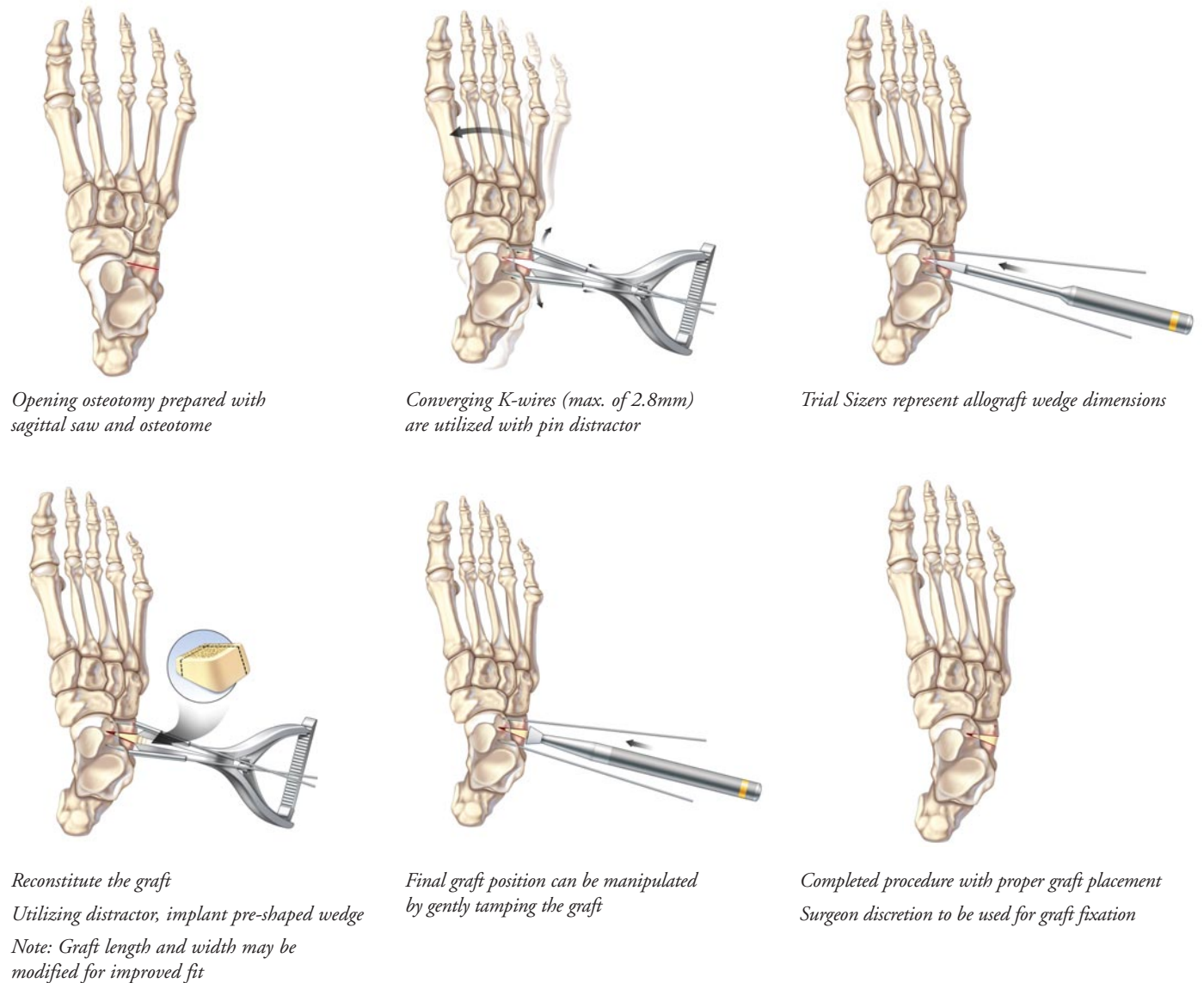

*Opening osteotomy prepared with sagittal saw and osteotome*

*Converging K-wires (max. of 2.8mm) are utilized with pin distractor*

*Trial Sizers represent allograft wedge dimensions*

*Reconstitute the graft  
Utilizing distractor, implant pre-shaped wedge  
Note: Graft length and width may be modified for improved fit*

*Final graft position can be manipulated by gently tamping the graft*

*Completed procedure with proper graft placement  
Surgeon discretion to be used for graft fixation*

## ORDERING INFORMATION

### EVANS WEDGE

Code	Size
403106	6mm
403108	8mm
403110	10mm
403112	12mm
403114	14mm
403116	16mm

### COTTON WEDGE

Code	Size
403205	5mm
403206	6mm
403207	7mm
403208	8mm

### ARCH WEDGE

Code	Size
403305	5mm
403306	6mm
403307	7mm
403308	8mm

### ALLOGRAFT WEDGE INSTRUMENTATION

Code	Description
903000	Instrument Set, Full
903001	Instrument Tray, Empty
903002	Osteotome, 16mm
903004	Osteotome, 10mm
903006	Mallet
903008	Distractor
903010	Bone Tamp, Large
903012	Bone Tamp, Small
903106	Evans Trial, 6mm
903108	Evans Trial, 8mm
903110	Evans Trial, 10mm
903112	Evans Trial, 12mm
903205	Cotton Trial, 5mm
903206	Cotton Trial, 6mm
903207	Cotton Trial, 7mm
903208	Cotton Trial, 8mm

## TISSUE SAFETY & INTEGRITY

MTF is a non-profit organization founded in 1987. Since its inception, MTF has recovered over 90,000 donors and distributed over 5 million grafts for transplantation while maintaining an exemplary safety record. Because of all of the safety steps instituted by the Board of Directors, Medical Board of Trustees, and Donation Board of Trustees, MTF has become the number one tissue bank in the nation and is one of the largest providers of grafts in the world.

- MTF employs the latest technologies such as Nucleic Acid Testing and conducts serological tests on all donors.
- MTF leads the industry in donor selection criteria and the standards exceed those set by the American Association of Tissue Banks (AATB) and most other tissue banks.

Available through:



ConMed Linvatec  
11311 Concept Boulevard  
Largo, FL 33773-4908  
800.237.0169 • Fax 727.399.5256  
To order call 800.433.6576  
or +1.732.661.0202  
[www.linvatec.com](http://www.linvatec.com)

Processed by:



Musculoskeletal Transplant Foundation  
125 May Street • Edison, NJ 08837  
732.661.0202 • Fax 732.661.2298  
[www.mtf.org](http://www.mtf.org)