



INTENDED USE

A disposable ULPA Filter for the management of surgical smoke for use with the Stryker® NEPTUNE® 2 and NEPTUNE® 3.

ULPA Surgical Smoke Replacement Filter

for use with the Stryker® NEPTUNE® 2 and NEPTUNE® 3.

Filter Installation Instructions:	
1	Examine all filters for damage during shipping and storage. Do not install any filter with visible signs of structural damage.
2	Remove protective cover from bottom of filter.
3	Insert the filter into the smoke evacuator filter compartment. Be sure that the filter is seated completely against the back of the filter chamber and the label that indicates "this side up" is facing upward.
WARNING	
The device is not intended for evacuation of fluid. If fluid is expected to be aspirated to the filter, fluid collection devices must be installed with the vacuum hose assembly. Failure to install a fluid collection device may cause filter blockage and/or electrical damage.	
Filter Removal Instructions:	
1	After the filter has been exhausted and requires changing, turn the smoke evacuation system off and disconnect any accessory tubing attached to the filter.
2	Pull the filter from the smoke evacuation system and dispose of in accordance with hospital policy. The filter may be disposed of or incinerated.
3	Refer to system manual for cleaning instructions.
WARNINGS	
Replace this filter as indicated by the system. Failure to change this filter may result in decreased efficiency and contamination of the electric motor, and sound absorbing media within the system, or non-operation of surgical smoke evacuator.	
The ULPA Replacement Filter should be considered bio-hazardous after use and should be disposed in the supplied bio-hazardous bag.	
The ULPA Replacement Filter and its corresponding system should only be used by a trained individual.	
CAUTIONS	
Federal law (United States of America) restricts this device to be used by, or on the order of a physician. Do not block either the tubing or the filter. If either becomes occluded or significantly restricted, the vacuum motor may overheat and cause the unit to fail.	

 Manufactured by:
Buffalo Filter LLC
5900 Genesee Street, Lancaster, NY 14086
U.S. toll-free 800.343.2324 · International: +1 716.835.7000
Fax: +1 716.835.3414
Email: bf.smoke.info@filtrationgroup.com
www.buffalofilter.com
903208 RevB 07/27/2017

		
		EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands
Not Made with Natural Rubber Latex		
Consult Symbol Glossary at www.buffalofilter.com		