



BEAMER AVEO™

Electrosurgical and Argon Platform



Advanced Energy, **Made Easy**

Experience an energy platform made just for you.
Schedule an evaluation today with your local CONMED Representative or
call 1-866-4CONMED so we can connect you. Use the QR code or visit
www.CONMED.com/BeamerAVEO for more resources.





BEAMER AVEO™



Advanced **V**oltage **E**nergy **O**utput

10,000 unique energy setting combinations; effect, interval, duration.


Your Energy, Your Way

Prefer to take control of your energy? Easily modify the tissue effect to the needs of your patient and precise standards, by customizing the following settings:

Tissue Effect (Coagulation Power)

Cut Duration (Cutting Power)

Cutting Interval (Pulse Rate)

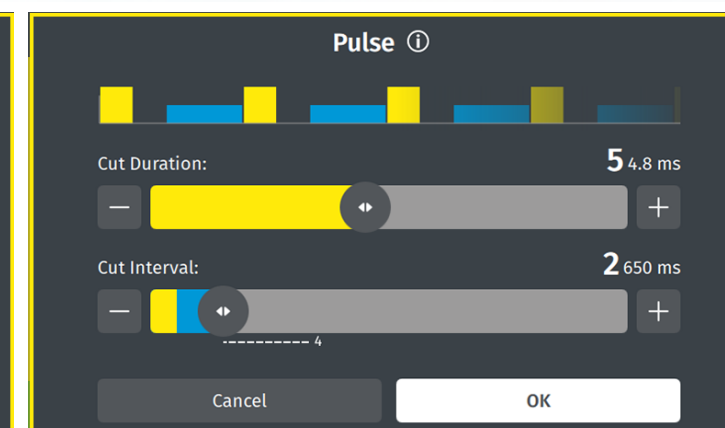

Master of One

This isn't a jack of all trades.

Unlike other generalist energy platforms designed for multiple specialties — which can add complexity, unnecessary steps and limit performance - this system was designed **just for you**.

Through collaboration with leading endoscopists and nurses, we developed a better, GI-focused energy system that includes all the features you need, and none you don't.

Advanced Energy, **Made Easy**

Advanced Endo Modes designed specifically for GI Therapeutic Endoscopic Procedures

EMR
ESD

POEM
ERCP

Hot Stent
Sphincterotomy

Multiple Argon Procedures
Full Thickness Resection



Tailor-Made *Performance*

26 GI Programs, Refined and Pre-Loaded

After extensive physician validation, we pre-loaded 26 GI-specific programs that enable you to easily achieve your optimal tissue effect.

Coagulation Performance

Coag dissect designed for effective hemostasis and third space dissection

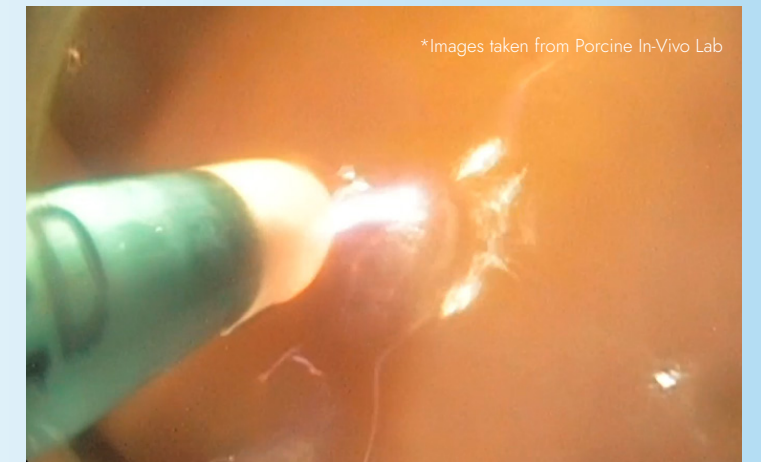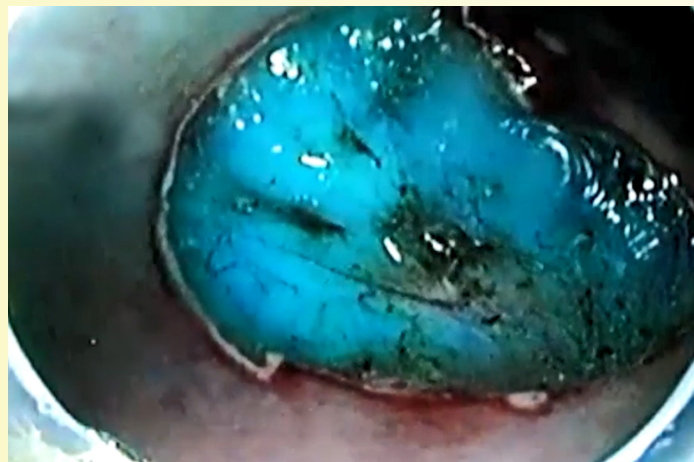

Cutting Performance

EndoMode Currents optimized for ERCP, ESD, POEM, EMR and Polypectomy

Argon Performance

Location specific programs — GAVE, AVM, Argon Esophagus/Right Colon/Rectum

Pure Cutting
Increasing Coagulation Effect

Deep Coagulation
Increasing Cutting Effect

Cutting with Coagulation

Pure Cut

Constant cutting current with minimal coagulation effect

Blend Cut

Intermittent cutting with intense coagulation

EndoMode D

Current designed for dissection type GI procedures (Sphincterotomy, ESD)

EndoMode R

Current design for resection type GI procedures (EMR, Polypectomy)

Coag Dissect

Fast effective coagulation current suitable for dissection with high hemostasis

Pure Coag

Tissue dissection properties and high hemostasis content

Forced Coag

Strong coagulation current with spark formation

Spray Coag

High-voltage coagulation current for non-contact coagulation

Gentle Coag

Coagulation current which avoids spark formation. Commonly used for tissue marking and small hemorrhages.

We've got this energy covered,
so you can focus your energy on what *matters*.



A Streamlined Energy Offering

Minimal Training Required

This system was designed to be intuitive, mimicking a smartphone touchscreen interface.

- Minimal touch points for physician-requested adjustments
- Visual aids simplify device operation
- Color-coded prompts ensure intuitive setup

A Place For Everything

Maximum benefits with minimal footprint saves valuable space

Modular Customizable Cart Setup


Height, options and accessories are customizable. Have your cart, your way.



BEAMER AVEO™

Filter Integrated Argon Probes

Beamer AVEO™ SideFire Probe

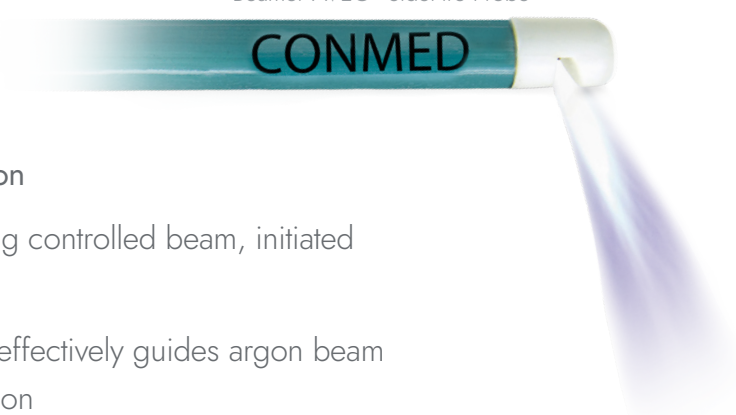

Ceramic Tip

- Designed to improve **visualization**
- **Non-Contact Coagulation** — long controlled beam, initiated without contacting mucosa
- **Targeted Therapy** —ceramic tip effectively guides argon beam for precise, controlled coagulation

Beamer AVEO™ Argon Probe



Fully Disposable

- Plug-and-play design
- No cables or accessories
- Filter Integrated





Financial Flexibility



Do you know your options?



Placement

Loan CONMED equipment with a disposable commitment



Save Space and Money

Unique design reduces storage needs and eliminates detergent shipping costs



Deferred Payment

Up to 90 days with credit approval



BEAMER AVEO™

Ordering Information

Description	Catalog Number
<i>Capital System Parts</i>	
Beamer Aveo™	
Electrosurgical Generator	AVEO-ESU
Argon Module	AVEO-ABC
Wired Foot Pedal	AVEO-1W
Cart (no tank holder)	AVEO-CART-PKG
Large Tank Holder (8.25"/20.9cm)	AVEO-TANK-LRG
Argon Pressure Reducer	CB200-A02
<i>Argon Disposables</i>	
Filter Integrated Argon Probe 1.8mm X 160cm	FIA-AVEO-1
Filter Integrated Argon Probe 1.8mm X 320cm	FIA-AVEO-2
Filter Integrated Argon Probe 2.3mm X 230cm	FIA-AVEO-3
Filter Integrated Argon Probe 2.3mm X 320cm	FIA-AVEO-4
Filter Integrated Argon Probe 3.2mm X 230cm	FIA-AVEO-5
Filter Integrated Argon SideFire Probe 2.3mm X 230cm	FIS-AVEO-3
<i>Warranty Options</i>	
Extended Warranty 2 years	AVEO-WARRANTY2
Extended Warranty 4 years	AVEO-WARRANTY4
Extended Warranty 6 years	AVEO-WARRANTY6

You have a budget — *we get that.*

Our flexible financial programs ensure you don't sacrifice innovation for financial limitations.



Experience an energy platform made just for you.
Schedule an evaluation today with your local CONMED Representative or
call 1-866-4CONMED so we can connect you. Use the QR code or visit
www.CONMED.com/BeamerAVEO for more resources.





CONMED Corporation
11311 Concept Blvd
Largo, FL 33773

Toll Free: 1-866-4CONMED
International: 727-214-3000
www.CONMED.com

customerexperience@conmed.com
internationalorders@conmed.com

