

## **BSPF030**

Laser and Electrosurgery  
Smoke Plume Prefilter

### **INSTRUCTIONS**

The **Buffalo Filter Laser and Electrosurgery Smoke Plume Prefilter** is designed to be a replacement filter (Buffalo Filter P/N BSPF030) for the Surgimedics® Surgifresh™ Prefilter (Surgimedics P/N 901202-000). This filter is used with all Surgimedics Surgifresh Smoke Filtration Systems: Surgimedics Mini and Surgimedics Plume-inator. This filter can also be used in ValleyLabs AirForce Smoke Evacuation System.

1. Turn unit off and unplug the system from electrical power.
2. Remove the prefilter from the top of the unit.
3. Dispose of the filter according to the protocol of your facility.
4. Install a Buffalo Filter Replacement Prefilter, BSPF030, by placing the prefilter securely into the top of the ULPA/Carbon Laser Smoke Plume Filter, part number 901009-010. If you are using an older version of the smoke evacuation unit, the prefilter can also be directly installed into the top opening of the smoke evacuator.
5. Proceed with the installation of an ULPA/Carbon Replacement Filter if the filter change indicator shows that the ULPA filter needs to be changed. See the instructions for the changing of the ULPA/Carbon Filter for more details.

**Buffalo Filter Replacement Prefilter (P/N BSPF030) is designed to be changed after every use.**

**For ordering information:  
Buffalo Filter®  
5900 Genesee Street  
Lancaster, New York 14086**

**(716) 835-7000  
(800) 343-2324  
(716) 835-3414 FAX**



Emergo Europe  
Molenstraat 15  
NL-2513 BH The Hague  
The Netherlands



900249REVB DFU BSPF030.doc