

## **BTLD001 AND BTLD004**

**TeleVac® Filter**

**Berchtold®/Buffalo Filter®**

**Reorder Number: BTLD004**

*For the filtration of smoke plume generated during surgical procedures*

### **INSTRUCTIONS**

#### **PRECAUTION:**

- Before installing or removing any filter, be sure that the smoke evacuator system motor is turned off.
- Install a fluid collective device (BF P/N SCC1001 or SCC1002) if excessive fluid is expected. Failure to install a fluid collection device could cause filter blockage and electrical damage

#### **DIRECTIONS:**

1. Install the filter into the round opening on the front of the smoke evacuation module of the Berchtold Televac. Assure proper seating of the filter. The gasket on the bottom of the filter must contact to the back plate of the filter housing.
2. Attach adapters or hose onto the filter inlet port. Assure that good sealing is achieved; install other adapters, wands, cones, tubing, etc. as required.
3. At the end of the recommended use time, remove the filter from the smoke evacuation system and dispose of in accordance with local codes. Used filters should be considered as potentially infectious waste.

#### **For ordering information:**

**Buffalo Filter®**

**5900 Genesee Street**

**Lancaster, New York 14086**

**[www.buffalofilter.com](http://www.buffalofilter.com)**

**716.835.7000**

**800.343.2324**

**716.835-3414 FAX**



Emergo Europe  
Molenstraat 15  
NL-2513 BH The Hague  
The Netherlands

