

BFE20009

Laser and Electrosurgery
Smoke Plume ULPA/Carbon Filter

INSTRUCTIONS

The Buffalo Filter "Edge Main ULPA/Carbon Filter" (P/N BFE20009) is designed to be a replacement for Edge Systems Corp main ULPA/Carbon "Big Boy" Filter (P/N 20009). This filter is used with Edge Systems Corp., the S-A-F-E System™ smoke evacuator, and used in conjunction with the Edge Prefilter (P/N BFE20004).

1. Turn unit off and unplug the system from electrical power.
2. Remove the prefilter from the top of the main ULPA/Carbon "Big Boy" Filter.
3. Lift/Remove the main ULPA/Carbon "Big Boy" filter from the Smoke Evacuator.
4. Dispose of the used prefilter and ULPA/Carbon "Big Boy" filter according to the protocol of your facility.
5. Install Buffalo Filter Replacement ULPA/Carbon Filter (BFE20009) by inserting the main cylinder, plastic base down, into the S-A-F-E System™ smoke evacuator.
6. Proceed with installation of a new prefilter. See the instructions for changing the prefilter for more details.

Buffalo Filter ULPA/Carbon Filter Replacement (P/N BFE20009) is designed for a maximum filter life of 25 hours depending on use. The main ULPA/Carbon Filter should be replaced when the pressure gauge on the smoke evacuator indicates the need for a filter change (see S-A-F-E System™ smoke evacuator users manual for more details).

For ordering information:

Buffalo Filter®

**5900 Genesee Street
Lancaster, New York 14086**

(716) 835-7000

(800) 343-2324

(716) 835-3414 FAX



Emergo Europe
Molenstraat 15
NL-2513 BH The Hague
The Netherlands



900263REVB DFU EDGE MAIN.doc