

BFE20004

Laser and Electrosurgery
Smoke Plume Prefilter

INSTRUCTIONS

The Buffalo Filter Edge Prefilter (P/N BFE 20004) is designed to be a replacement for Edge Systems Corp Prefilter (P/N 20004-05). This filter is used with Edge Systems Corp., the S-A-F-E System™ smoke evacuator, and used in conjunction with the ULPA/Carbon “BIG BOY” Filter (P/N 20009 and BFE20009).

1. Turn unit off and unplug the system from electrical power.
2. Remove the Prefilter from the top of the main ULPA/Carbon “Big Boy” Filter
3. Dispose of the used prefilter according to the protocol of your facility.
4. Install Buffalo Filter Replacement Prefilter (P/N BFE20004) by placing the prefilter securely onto the top of the main ULPA/Carbon “Big Boy” Filter.
5. **If** the pressure gauge on the S-A-F-E System™ smoke evacuator shows that the ULPA/Carbon filter needs to be changed, **Then** proceed with installation of an ULPA/Carbon “Big Boy” replacement filter (see S-A-F-E System™ smoke evacuator users manual). See the instructions for changing the ULPA/Carbon Filter for more details.

SINGLE USE ONLY. Buffalo Filter Replacement Prefilter (P/N BFE20004) is designed to be changed after every use.

For ordering information:
Buffalo Filter®
5900 Genesee Street
Lancaster, New York 14086

(716) 835-7000
(800) 343-2324
(716) 835-3414 FAX



Emergo Europe
Molenstraat 15
NL-2513 BH The Hague
The Netherlands



900264REVB DFU EDGE PF.doc