



Hercules[®] XE

Extended Eyelet Knotless Anchor
for BioBrace[®]

PROCEDURE KIT NOW AVAILABLE



Extended eyelet specifically
designed for the 5mm x 250mm
BioBrace[®] Reinforced Implant

SURGICAL TECHNIQUE



Hercules® XE

Extended Eyelet Knotless Anchor for BioBrace®

Product Information

Product Description

The Hercules® XE Knotless Anchor System provides a radiolucent PEEK implant for use in a range of soft tissue fixation applications. The extended eyelet (XE) allows for use with BioBrace® 5 x 250mm, which is a biocomposite soft tissue scaffold intended for use in a range of surgical procedures to reinforce soft tissue where weakness exists.


The BioBrace® Reinforced Implant is sold separately.

Important: BioBrace® is not intended to be a standalone construct, but an augment to other tissue/ligament repair procedures. Please refer to the BioBrace® Instructions for Use for applicable indications.

Indications

The Hercules® Knotless XE Anchors are intended for use in the following applications for:

- **Shoulder:** Rotator Cuff, Bankart and SLAP lesion repair, Biceps tenodesis, Acromio-Clavicular separation and Deltoid Repair, Capsular shift and Capsulolabral reconstruction
- **Foot and Ankle:** Lateral and medial ankle instability, Achilles tendon and Metatarsal ligament repair, Hallux Valgus and Midfoot reconstruction
- **Knee:** Medial collateral and Lateral collateral ligament repair, Patellar tendon, and Posterior oblique ligament repair
- **Hand/Wrist:** Scapholunate ligament, Radial collateral ligament and Ulnar collateral ligament reconstruction
- **Elbow:** Biceps tendon reattachment, Tennis elbow repair, Ulnar and Radial collateral ligament reconstruction.

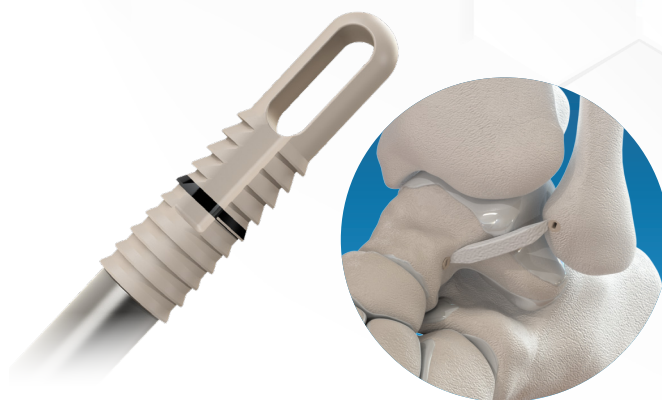


Contraindications

General contraindications for the use of these implants for joint reconstruction include:

- Significant bone demineralization
- Inadequate neurovascular status
- Poor blood supply, which may impair the healing process
- Inadequate skin or musculotendinous system
- Physiologically or psychologically unsuitable patient
- Surgical procedures other than those indicated
- Possibility for conservative treatment
- Bone, musculature, tendons, or adjacent soft tissue compromised by disease, infection, or prior implantation, which cannot provide adequate support or fixation for the prosthesis.
- Known allergies
- Diabetes
- Active infection to materials
- Patients with high level of activity
- Pathological conditions of the soft tissue which would compromise secure fixation


Hercules® XE Extended Eyelet Knotless Anchor for BioBrace®

Surgical Technique | ATFL Repair

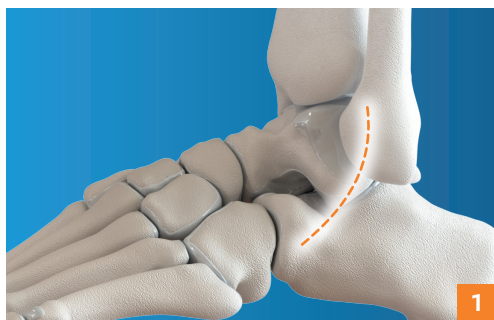


Site Preparation

Expose and prepare the site following standard procedures (**Figure 1**), using available equipment and instrumentation, per surgeon discretion and expertise. Implantation of the Hercules® XE Knotless Anchors with BioBrace® should be performed as an augment to tissue/ligament repair procedures. Identify good quality bone for effective Anchor placement and determine appropriate Anchor size and corresponding Instrument Kit for the indicated area.

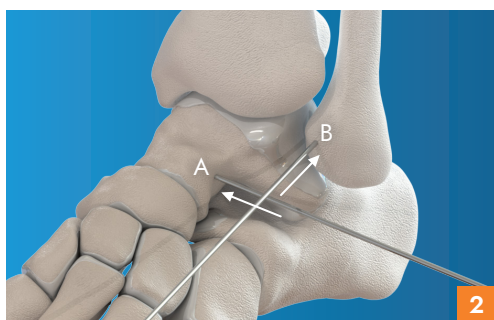

Insert Guide Wires

Insert the first Guide Wire (Ø.062mm x 9" length) perpendicular to the bone surface into the talus (**A**) at the insertion point of the anterior talofibular ligament (ATFL). Then insert the second Guide Wire into the fibula (**B**) at the origin of the ATFL (**Figure 2**).

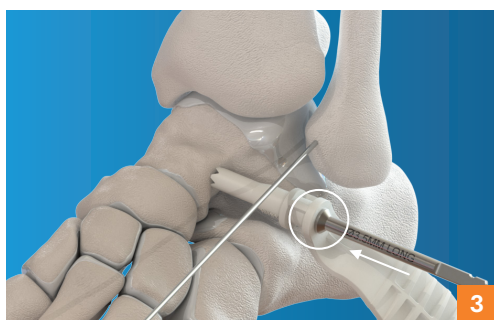

Create Pilot Holes

Starting with the talus, place the Cannulated Drill and Drill Guide over the Guide Wire (**Figure 3**). Advance the Drill until the drill stop reaches the Guide, then remove the Drill and Guide Wire.


Repeat for the fibula.

Note: The Guide Wires may be left in the site for reference until ready to insert the Anchors, so that the pilot holes are not mislocated.

Where softer bone quality is present, the 3.2mm Drill Kit (A04 S0132) can be used for the 3.5mm Hercules® XE Knotless Anchor.

Thread BioBrace® into the Anchor

Using the Suture Threader attached to the Driver assembly, pass the BioBrace® Reinforced Implant through the extended eyelet of the Hercules® Knotless XE Anchor and pull the Suture Threader through the eyelet until one end of BioBrace® is made available opposite the point of entry (**Figure 4**).

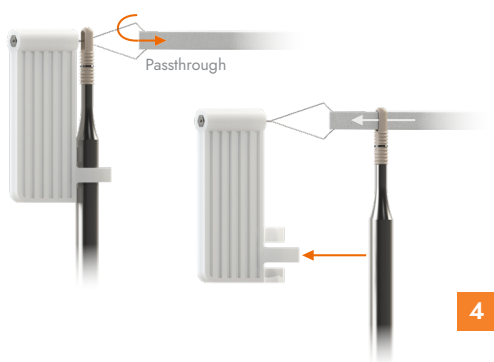


Alternative Method:

If needed, cut the end(s) of BioBrace® to a shallow taper. With the Anchor loaded onto the driver, thread the tapered end of BioBrace® manually through the eyelet of the Anchor and grab with fingertips or forceps on the opposite side of the eyelet and pull through (**Figure 5**).




Hercules® XE Extended Eyelet Knotless Anchor for BioBrace®

Surgical Technique | ATFL Repair

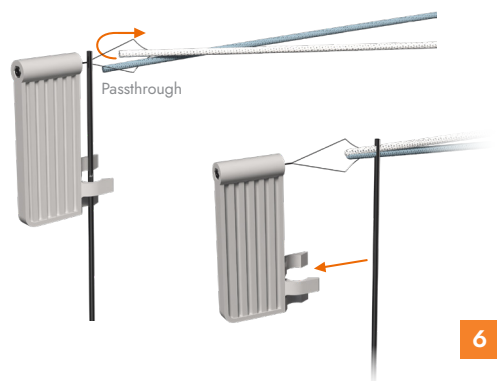

Thread Suture into the Guide Wire


Using the Suture Threader, pass the suture tails through the eyelet of the Guide Wire and pull the Suture Threader through the eyelet (**Figure 6**). Pull the Guide Wire through until the tails are made available through the prepared bone tunnel opposite the point of entry.

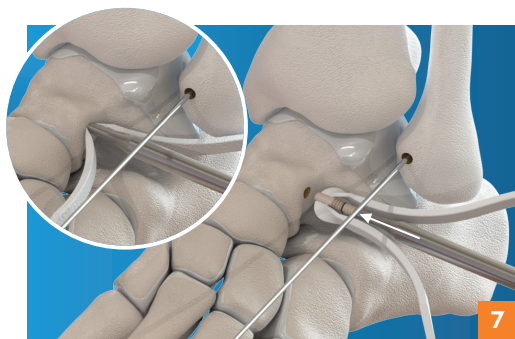

Talus Anchor Insertion

Ensure that only a portion of the BioBrace® material is pulled through the eyelet of the initial Anchor for the Talus, allowing the majority of the material to remain available for placement in the fibula. Using the Driver assembly, insert the Knotless Anchor into the talus through the prepared hole and advance the Anchor with a mallet until fully seated and flush to the bone surface. Pull the Driver away from the Anchor to disengage (**Figure 7**).

Note: Do not apply clockwise quarter turn for use with BioBrace®.

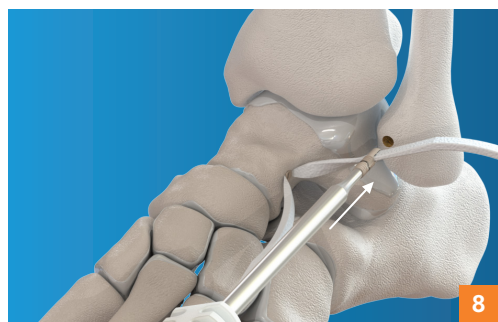


Fibula Anchor Insertion

Using the longer, loose tail, thread a second Anchor with the BioBrace® and manually tension the BioBrace®, via surgeon-preferred method, prior to final seating of the second Anchor (**Figure 8**). Trim away any excess BioBrace® (**Figure 9**).


Close the Site

Close the site via surgeon-preferred method.

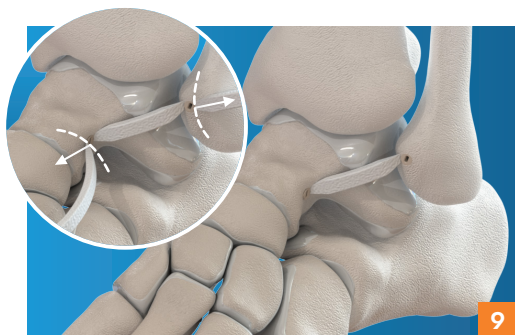

Hercules® XE

Extended Eyelet Knotless Anchor for BioBrace®

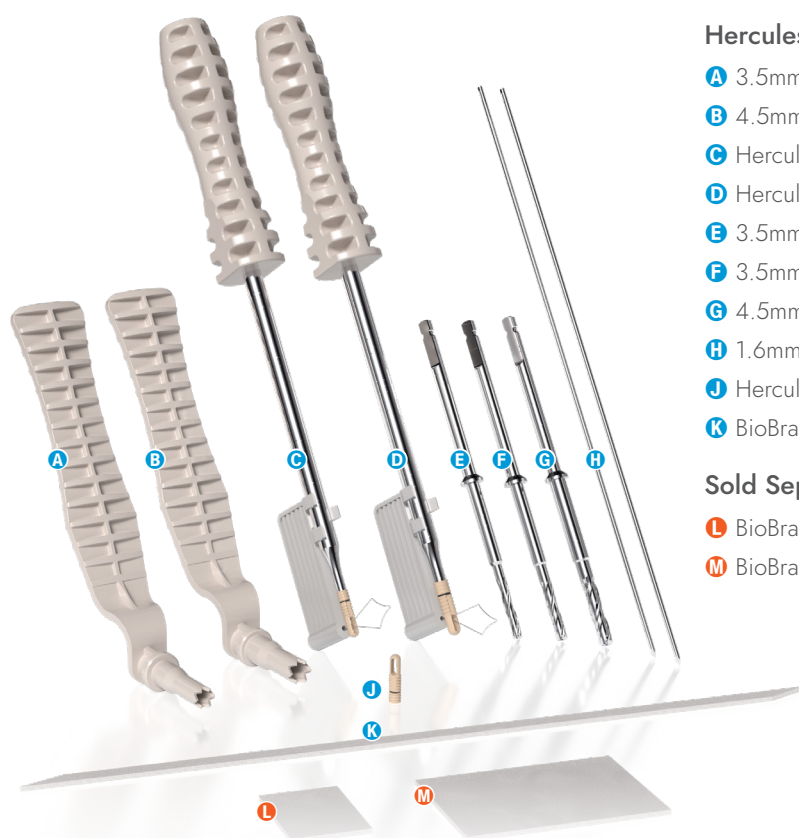

Hercules® XE Sterile Procedure Kit | A40 SPB01

- A** 3.5mm Soft Tissue Protector
- B** 4.5mm Soft Tissue Protector
- C** Hercules® XE 3.5 x 15mm Anchor & Assembly
- D** Hercules® XE 4.5 x 18mm Anchor & Assembly
- E** 3.5mm Cannulated Drill Bit, 3.2mm
- F** 3.5mm Cannulated Drill Bit, 3.6mm
- G** 4.5mm Cannulated Drill Bit, 4.7mm
- H** 1.6mm Guide Wire, 8" (Qty. 2)
- J** Hercules® XE 3.5 x 15mm Anchor
- K** BioBrace® FA 5 x 250mm Implant

Sold Separately:

- L** BioBrace® FA 23 x 30mm Implant BB23X30-FA
- M** BioBrace® FA 40 x 60mm Implant BB40X60-FA

Also Available Individually:

Hercules® Knotless Anchor Implant Kits - Sterile

DESCRIPTION

Hercules® XE Knotless Anchor, 3.5mm
Hercules® XE Knotless Anchor, 4.5mm
Hercules® Knotless Anchor, 5.5mm

CATALOG NO.

A40 SP135
A40 SP145
A40 SP055

Hercules® Knotless Anchor Instrument Kits - Sterile

DESCRIPTION

Hercules® XE Knotless Instrument Kit, Size 3.5mm, 3.2mm Drill
Hercules® XE Knotless Instrument Kit, Size 3.5mm
Hercules® XE Knotless Instrument Kit, Size 4.5mm, 4.7mm Drill
Hercules® Knotless Anchor Instrument Kit, 5.5mm

CATALOG NO.

A04 S0132
A04 S0135
A04 S0147
A04 S0055

BioBrace® FA Implants - Sterile

DESCRIPTION

BioBrace® FA 5mm x 250mm
BioBrace® FA 23mm x 30mm
BioBrace® FA 40mm x 60mm

CATALOG NO.

BB5X250-FA
BB23X30-FA
BB40X60-FA

Guide Wires - Sterile

DESCRIPTION

Guide Wires (2), S. Trocar, w/ Eyelet, .062" x 8", Single-Use

CATALOG NO.

C01 S2268



In2Bones USA
7750A Trinity Rd.
Cordova, TN 38018
844 602 6637
www.conmed.com/in2bones

In2Bones SAS
28, chemin du Petit Bois
69130 Ecully
France
+33 (0)4 72 29 26 26

CONMED Corporation
11311 Concept Blvd.
Largo, FL 33773
USA

Some products not available in all markets. This document provides information regarding medical devices and instruments in surgical procedures. It is not medical advice and healthcare professionals should use their own judgment before using to treat a particular patient. Healthcare professionals should be trained in the use of such devices before surgery and should always refer to the package insert, product label and/or instructions for use, including the instructions for cleaning and sterilization (if applicable), before using any product.