

Blades and Burs

Pocket Reference Guide



Ultra Dual-Purpose Blades by Procedures

	PROCEDURE	ULTRA DUAL-PURPOSE BLADE
SHOULDER	Sub-Acromial Decompression	UltraFRR® Stealth™
	Rotator Cuff Repair	UltraGATOR® or UltraFRR® Stealth™
	Instability Repair	UltraFRR® Stealth™
	Biceps Repair	UltraGATOR® Stealth™
	Bankart/SLAP	UltraFRR® Stealth™
KNEE	Meniscectomy	UltraTIGER® or UltraGATOR® Stealth™
	ACL Reconstruction	UltraGATOR® Stealth™
	PCL Reconstruction	UltraGATOR® Stealth™
	Cartilage/Osteochondral Debridement	UltraTIGER® or UltraFRR® Stealth™
HIP	Hip Preservation	UltraGATOR® HPS
		UltraFRR® HPS

HUB COLOR:

UltraTIGER®  UltraGATOR®  UltraFRR® 

Ultra Dual-Purpose Blades Aggressiveness

	LEAST —————> MOST			
TISSUE	Ultracut®	UltraTIGER®	UltraFRR® Stealth™	UltraGATOR® Stealth™
BONE	Ultracut®	UltraTIGER®	UltraGATOR® Stealth™	UltraFRR® Stealth™

HUB COLOR:

Ultracut®  UltraTIGER®  UltraGATOR®  UltraFRR® 

- The Ultra Dual Purpose blades require shaver consoles D4000™/D4000A™ with software version 2 or newer.
- For ideal tissue and bone resection set the D4000™/D4000A™ settings to 1,500 rpms to 4,000 rpms in oscillate mode and 8,000 rpms to 12,000 rpms in forward/reverse mode.

Standard Blades by Procedures and Tissue

	PROCEDURE	TISSUE	BLADE
SHOULDER	Sub-Acromial Decompression	CA Ligament, Bursa	Gator® Stealth™
	Rotator Cuff Repair	Labrum, CA Ligament, Bursa	Great White® or Gator® Stealth™, Lanza™
	Instability Repair	Labrum	Gator® or Full-Radius Resector Stealth™, Lanza™
	Biceps Repair	Biceps	Great White® Stealth™ or Cuda®
	Bankart/SLAP	Labrum	Gator® or Full-Radius Resector Stealth™, Lanza™
KNEE	Meniscectomy	Meniscus	Gator® Stealth™, Lanza™
	ACL Reconstruction	ACL, Synovium, Scar Tissue	Great White® Stealth™
	PCL Reconstruction	PCL	Great White® Stealth™
	Cartilage/Osteochondral Debridement	Cartilage	Gator® Stealth™, Lanza™
HIP	Soft Tissue Removal	Labrum	Great White® Stealth™ or Cuda®
	Cartilage/Osteochondral Debridement	Cartilage	Great White® Stealth™ or Cuda®

HUB COLOR:

Full-Radius Resector  Gator®  Cuda®  Great White®  Lanza™ 

Blades Aggressiveness

	LEAST → MOST					
TISSUE	Bursa	Cartilage	Meniscus	Biceps	ACL	
SOFT TISSUE	Labrum		Synovium		Rotator Cuff	
BLADE	Tiger®	Lanza™	Full-Radius Resector Stealth™	Gator® Stealth™	Cuda®	Great White® Stealth™

HUB COLOR:

Tiger®  Full-Radius Resector  Gator®  Cuda®  Great White®  Lanza™ 

Burs by Procedures and Bone

	PROCEDURE	BONE	BUR
SHOULDER	Sub-Acromial Decompression	Acromion & Bony Spur	Vortex®
	Rotator Cuff Repair	Bleeding Bone Bed	Spherical or Oval
KNEE	Notchplasty	Articular Cartilage Surface	Spherical or Oval
	Cartilage/Osteochondral Debridement	Articular Cartilage Surface	Spherical or Oval
HIP	CAM Lesion	Articular Cartilage Surface, Cortical Bone	Spherical or Oval
	Pincer Lesion	Articular Cartilage Surface, Cortical Bone	Spherical or Oval
	Cartilage/Osteochondral Debridement	Articular Cartilage Surface	Spherical or Oval

HUB COLOR:

Polishing 

Spherical 

Oval 

Vortex® 

Burs Aggressiveness

		LEAST → MOST			
BONE	Articular Cartilage Surface	Bleeding Bone Bed	Acromion	Cortical Bone	Bony Spurs/Osteophytes
BUR	Polishing	Spherical	Oval	Vortex®	

HUB COLOR:

Polishing 

Spherical 

Oval 

Vortex® 



Blades and Burs

Pocket Reference Guide

525 French Road
Utica, NY 13502

Local 315-797-8375
Toll Free 800-448-6506

CONMED.com
info@mail.CONMED.com