


LaparoVue® Scope Warmer

SET-UP GUIDE

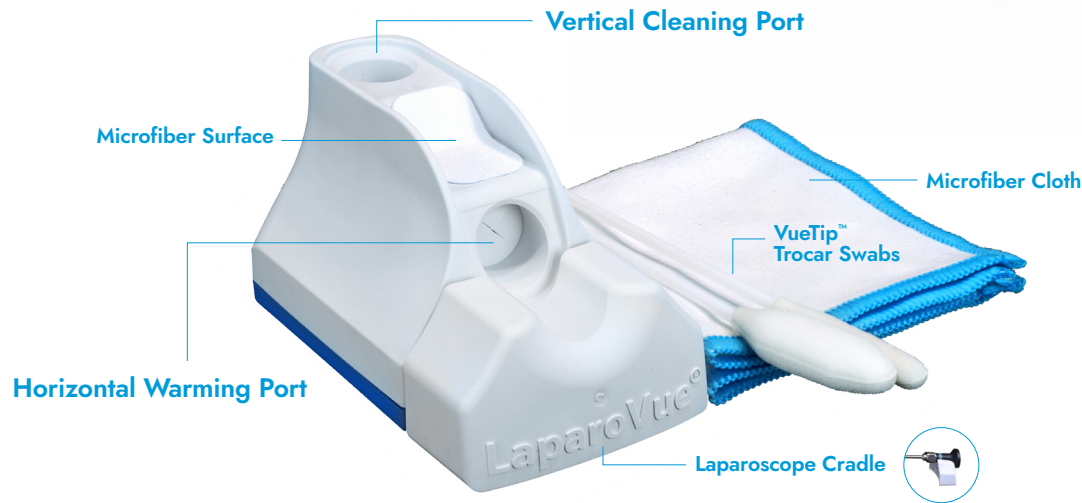


The 3 P's: **PULL, POP, PLUG**

Step 1

PULL the tab on the base of the unit to activate, a blue activation light will appear. **For best results, pull tab while setting case up.**

- The blue light turns off after 2 hours to preserve battery life for warming; the device still functions/warms for up to 5 hours.

PULL the scope cradle apart from the unit and use while warming the scope. This allows the laparoscope to rest level in the horizontal port.

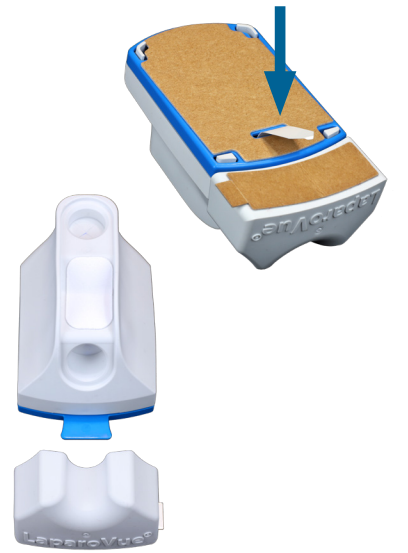

Step 2

POP the surfactant cup film in the vertical cleaning port using the proximal end of a trocar swab and move tip in a circular motion to open the film wide enough for the laparoscope.


- Don't use the laparoscope to puncture to prevent damage to the scope.



Step 3

PLUG the laparoscope fully into the horizontal warming port before the start of the case.

- Allow a minimum of 5 minutes to warm, 20 minutes recommended for best results.

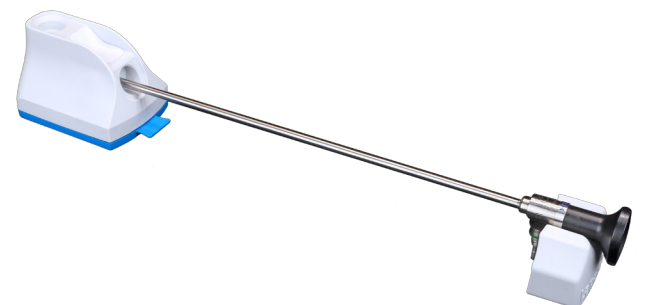

Notes

- LaparoVue is safe for use with robotic scopes.*
- When cleaning the trocar with VueTip swabs:
 - Make sure the trocar does not have any sharp edges that can snag the swabs.
 - Do not extend foam head beyond distal end of trocar.
 - It's recommended to use small swab on 5-8mm trocars and large swab on 10-12mm trocars.