



PadPro[®] MFE Pads

Recommendations for Skin Preparations for MFE Electrodes

Skin preparation and MFE pad placement can make a difference in how electrodes function. CONMED recommends the following when it comes to skin prep and placement of their PadPro[®] MFE pads. In addition, hospital protocol should be followed before following the below steps:

1. Ensure that skin is clean and dry.
2. If necessary, remove excess hair by clipping.
3. Do not apply any substance to the skin surface that will leave a residue, i.e. lotions, oils, etc.
4. Apply electrode to the patient by rolling the electrode from top to bottom. Ensure that the surface area of the electrode is in contact with the skin.
5. Follow your hospital protocol, or manufacturer's recommendations, when it comes to pad placement on patient.

If you have any further questions about CONMED PadPro[®] MFE pads, please contact your local Territory Manager, or call customer service at **1-800-237-0169**.

You may also find additional Instructions For Use online at
[CONMED.com/products/cardiology-and-critical-care](https://www.conmed.com/products/cardiology-and-critical-care)