



Spectrum[®] MVP[™]

OUR MOST VERSATILE SUTURE PASSER

- For use in instability, rotator cuff repairs, ramp lesions, and ACL avulsions
- Unique "Single-pass" device through one portal
- Ergonomic handle
- Pass with ease with MVP[™]



**INSTABILITY
AND RAMP
LESIONS**

Steps to Simple Suture Passing

BLUE-GREY-BLUE-GREY-

Hold the **BLUE** as you come back through.



BLUE

Basic Technique

GREY

Spectrum® MVP™ Suture Passer

The Spectrum® MVP™ Suture Passer offers surgeons ease during suture passing. Its basic technique allows for passing and retrieving with one device, through one portal. Spectrum® MVP™ brings versatility and efficiency to your OR with this simplified suture passing option.


Basic Technique



Create a small loop at the tip of the post strand of suture and place it in the Suture Guide Slot




Insert Spectrum® MVP™ to the surgical site, pass the suture hook through the desired tissue and deploy the Suture Loop by sliding the BLUE switch forward to the lock position.

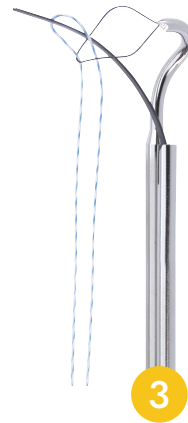

Pass the suture through the suture loop by sliding the GREY Suture Guide switch forward.



After the suture is passed through the suture loop release the BLUE Suture Loop switch to capture the suture. The spring assisted switch will close on the suture.

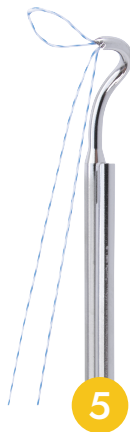

Slide the GREY Suture Clip switch back to the retracted position.



With your finger/thumb on the BLUE switch retract the suture hook through the tissue and the suture is passed.

**Technique Tip – During arthroscopic procedure, keep slight tension on the suture and hold extra suture to the side in the cannula*

ORDERING INFORMATION

Catalog No. Product Description Size

IMPLANTS

C6400	Spectrum® MVP™, Medium Crescent Hook, Disposable Packaged Sterile, 3/box	
C6410	Spectrum® MVP™, 45° Right Hook, Disposable Packaged Sterile, 3/box	
C6420	Spectrum® MVP™, 45° Left Hook, Disposable Packaged Sterile, 3/box	
C6400EL	Spectrum® MVP™, Medium Crescent Hook, Extended length, Disposable Packaged Sterile, 3/box	
C6410EL	Spectrum® MVP™, 45° Right Hook, Extended length, Disposable Packaged Sterile, 3/box	
C6420EL	Spectrum® MVP™, 45° Left Hook, Extended length, Disposable Packaged Sterile, 3/box	

ACCESSORIES

C6105	Crochet Hook	
C6112	Loop Handle Knot Pusher	
GU1009F	Katana™ High Strength Suture Cutter, 4mm Diameter, Straight	
11.1001	Grasping Forceps, 3.4mm Diameter, Straight with Ratchet	
16.1018	Suture Retrieval Forceps, 3.4mm Diameter	
25.50014	Liberator™ Knife	
25.50016	Rasp Liberator™ Knife	
HR713W	1.3mm Hi-Fi® Ribbon 40" Strand, no Needle (White/Blue)	
HR713B	1.3mm Hi-Fi® Ribbon, 40" Strand, no Needle (Blue)	
HT7150	2mm Hi-Fi® Tape (White/Black)	
HT7110	2mm Hi-Fi® Tape (Blue)	
H6100	Hi-Fi® Suture, Single 40" Strand, Blue No Needle	
H5120	Hi-Fi® Suture, Single 36" Strand, White-Blue Co-Braid, No Needle	
H5150	Hi-Fi® Suture, Single 36" Strand, White-Black Co-Braid, No Needle	

